

KIDS SINGLE-USE 3-PLY MASK



Suitable for Ages 6-12

BYD CARE 

SOFT & EASY TO BREATHE

High filtration of pathogen

BFE \geq 95%



Application

Designed for children between the ages of 6 - 12 to cover the mouth, nose and chin, the mask provides a physical barrier to block exhalation or ejection of pollutants from mouth and nose, and to reduce breathing in airborne particles.

*Picture is for reference only, actual product may vary.



Product Introduction

BYD Kids Single-use 3-Ply Mask consists of three layers of nonwoven material:

Outer Layer
Polypropylene spunbond nonwoven fabric

Middle Layer
Polypropylene melt-blown nonwoven

Inner Layer
Polypropylene spunbond nonwoven fabric

MATERIALS

Inner layer Polypropylene spunbond nonwoven fabric


Middle layer Polypropylene melt-blown nonwoven

Outer layer Polypropylene spunbond nonwoven fabric

Ear loops Polyester/nylon spandex blend

Nose clip Plastic

One-time use only. Expiration Date: 1 year after production

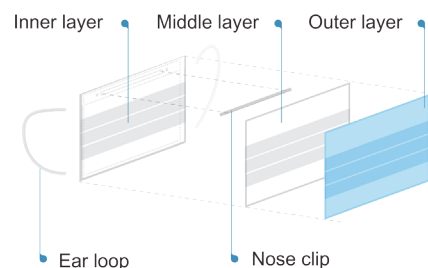

KIDS SINGLE-USE 3-PLY MASK



Suitable for Ages 6-12

Instructions for use



1. Wash your hands before touching the mask.



2. Ensure the colored side of the mask faces outwards, the nose clip upwards, and secure the ear loops on each side.



3. Bend the nose clip to match the shape of the nose to prevent unfiltered air from entering.



4. Pull the bottom of the mask under your chin to ensure a secure fit.

Parameter of BYD Single-use Face Mask

Name	Parameter
Product Name	BYD Kids Single-use 3-Ply Mask
Material	Polypropylene spunbond nonwoven fabric, polypropylene melt-blown nonwoven, plastic strip nose clip, polyester/nylon spandex blend ear loops
Product model	FE2411X
Size	145 mm × 95 mm
Expiration date	1 year after production
Specifications	10pcs/bag; 50pcs/box; 2000pcs/case
Storage	Store in a cool place, dry place, keep away from direct sunlight.
Colour	■ Blue/ ■ Pink

WARNING

1. The mask does not eliminate the risk of contracting any disease or infection.
2. Failure to properly use and maintain this product could result in illness or even death.
3. Only for one-time use, please destroy and dispose according to regulations after use.
4. Please use within the validity period.
5. DO NOT use if package is damaged.
6. People who are allergic to nonwovens should use with caution.
7. Children with dyspnea are not recommended to wear masks. Please follow doctor's advice or wear other respiratory protective equipment.
8. Children should wear a mask only under adult supervision and make sure the mask is worn correctly.
9. Children should not play or exercise at moderate or intense levels while wearing a mask. If you notice breathing difficulties or skin allergies appear while wearing the mask, please take it off immediately and seek for medical attention if necessary.
10. Masks should be stored in a cool and dry place.
11. Reusing or washing the mask is not recommended.
12. Used masks should not be exchanged with others.
13. Please refer to the instructions before use.



Limitations of Use

If the mask becomes damaged, soiled, or breathing becomes difficult, leave the contaminated area and replace and refit with a new mask.

All information presented in this document is based on the latest data available at the time of printing. BYD reserves the right to make changes at any time without prior notice.

Approval and Standard

GB/T 38880-2020



Manufactured by Xi'an BYD Electronics Co., Ltd.

Address: No.2 Yadi Road, New Industrial Park, Xi'an High-tech District, Xi'an, China

Tel: +86-29-88889999